Ganz genau.

HORICO DENTAL

INNOVATIONS / NEUHEITEN

2025

ORALCHIRURGIE

Oral surgery • Chirurgie buccale • Cirugía oral

Kit Osteotomie

(Dr. F. Faustini, Kit SKY1)

SKYLINE BONE
by Dr. F. Faustini

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Osteotomy is an oral surgery procedure in which teeth are exposed and removed.

EN **The kit contains four efficient instruments with handpiece shank in a practical autoclavable drill stand:**

- 1 x Lindemann bur for cutting and removing tooth roots.
- 2 x titanium nitride coated (TIN) carbide burs, ideal for reducing the alveolar bone and creating space for the placement of multiple abutments. The gold-coloured coating makes the surface of the burs smoother - this is gentler on the bone substance. It also protects the cutting edges from wear.
- 1 x diamond bur for smoothing / finishing of sharp edges.

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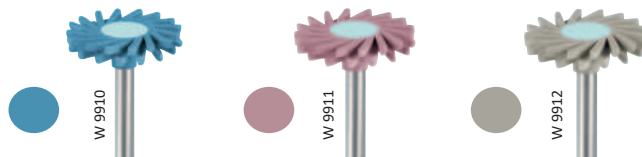
Designed for Zirconia and Lithium-(Di)-Silicate, also outstanding on all other dental preparations.

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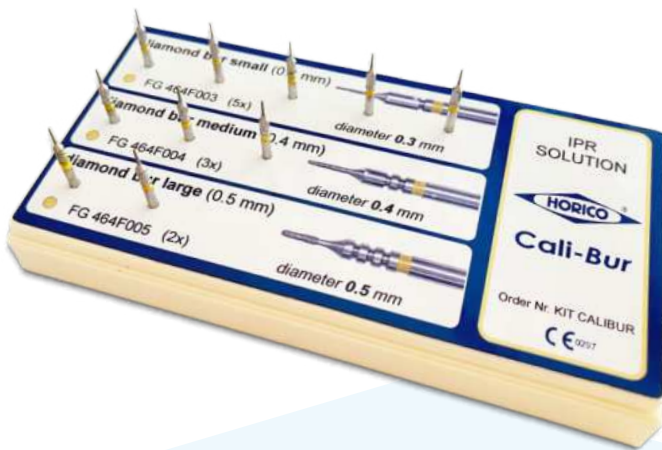
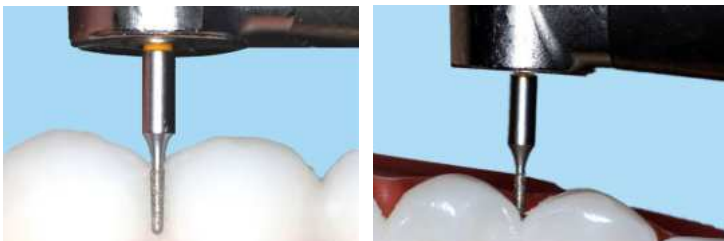
EN IPR diamond bur - for efficient space creation in the case of malaligned teeth. The innovative Cali-Bur burs have a slender shape in three diameters. The kit contains 10 burs: 5 x 0.3 mm, 3 x 0.4 mm and 2 x 0.5 mm.

Application: Proceed step by step, starting with the smallest diameter 0.3mm. Use with water cooling and throat pack / rubber dam. Swipe bur – do not force.

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EN The kit contains 10 instruments for orthodontics:

- For **debonding**:

- 1 debonding bur for gentle removal of adhesive, 2 polishers

- For **stripping (IPR)**:

- Widening of the approximal gap: 1 serrated steel strip 'PAUL'
- For approximal enamel reduction: 4 'PAUL' diamond strips in various thicknesses (yellow 0.1mm, red 0.14 mm, blue 0.17 mm, green 0.2 mm), coated on one side, as well as 2 thin stripping diamond burs

Step-by-step procedure for stripping: Start with the white strip holder to break the contact point. Then use the yellow strip holder to reduce the enamel, then continue with red, blue and green until you have reached the desired width.

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EN For zirconia protheses the dental technician needs skilled craftsmanship and first-class rotary instruments. This kit with 11 instruments for both green state zirconia and sintered zirconia helps with this.

- For green zirconia: three diamond burs and two tc cutters to reproduce the dental anatomy as accurately as possible, also for the cut-back technique
- For sintered zirconia: two green stones for adjusting and four polishers for a brilliant high lustre.

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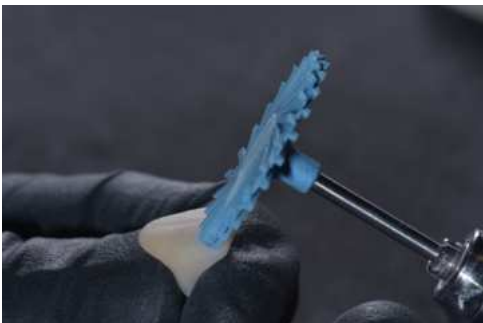
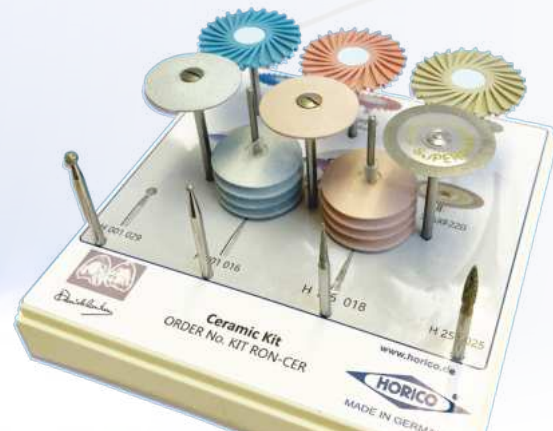
EN The RON-CER kit was developed for dental technicians who want to reproduce nature down to the smallest detail quickly and easily. Suitable for all types of ceramic. It contains:

- four diamond burs and a diamond disc for the life like reproduction of macro- and microstructures in dental restorations.
- a carefully selected range of polishes to create a high-gloss surface so that the ceramic can unfold its full brilliance. This includes the three-stage LAMELLO® polishing system with elastic lamellae for a perfect high lustre.

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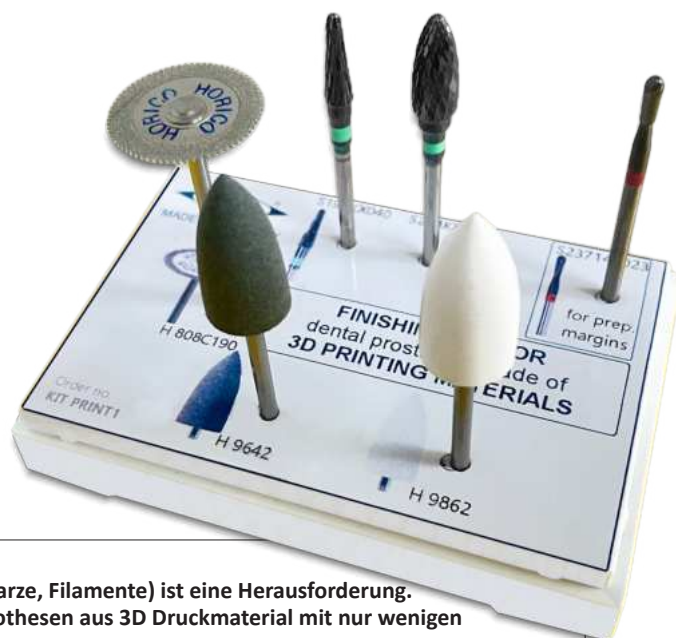
EN Finishing prostheses made of 3D printed materials (printing resin and filament) is a challenge. This newly developed kit enables the finishing of prostheses made of 3D printed material with just a few instruments. Kit contains 6 instruments:

- Diamond disc H 808C190: Easy removal of support structures / cutting out the prosthesis
- TC cutter S194KX040 / S274KX060: Ideal for eliminating support residues and grinding the prosthesis.
- Polisher H 9642: For easy removal of surface roughness.
- Polisher H 9862: For a beautiful, smooth surface
- TC cutter S237140023: For finishing at the preparation margin

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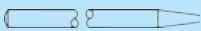
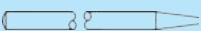
EN Special polisher (2-coloured) for the safe and efficient finishing of CAD/CAM milled structures of zirconia before sintering. Leaves a surface that makes post-sintering finishing superfluous.

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EN Polisher for the safe and efficient finishing of the edges and interdental spaces of prostheses milled from presintered zirconia blanks. Does not damage the edges of CAD/CAM milled structures even if the hardness of the blank is low. Leaves a very smooth surface.

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